



LANCET (HEMOLANCE DEVICE)



Escambia County, Florida - ALS/BLS Medical Protocol

Hemolance device.

- a. Release Needle:

Twist and pull to remove protective cap before use.

- b. Press:

Place the Hemolance lancet against the finger and then press and release the blue or green button. The needle retracts automatically when pressure is released.

- c. Dispose:

Dispose of lancets in appropriate sharps container or according to local disposal ordinance.



Approved: 10/01/2011
Version: 3.1

Page 1 of 1

Approved by:


Charles Neal, D.O. Medical Director